

Nasogastric (NG)Pad

About: These are for children and adults who are tube fed or need IV or NG medication, and are used to help prevent rubbing of the tube against the skin.


Materials:

- Top/ outer layer is patterned 100%cotton, flannel or flannelette
- Middle layer is bamboo or thin 100% cotton wadding
- Inner layer soft non pile towelling or flannelette (goes next to the skin)

Directions:

- Cut the below pattern (or a circular shape with a diameter of 7.5 cm is also suitable) from all three fabrics
- Ensure the central cut (indicated by the dotted line) is cut through all three layers.
- Place the wadding in the centre of the two fabrics, with the patterned and face sides of the fabrics facing outwards on either side of the wadding.
- Sew (overlocking or machine stitching) around the outside of the shape, and either side of the cut split.
- If you have access to a device, press studs can be applied to either side of the of the split, otherwise returning the completed item to Get Scrubbed is fine and we will organise to have the press studs added to the NG Pad (see example of completed item below)

Pattern: Hexagon or circle 7.5cm in diameter

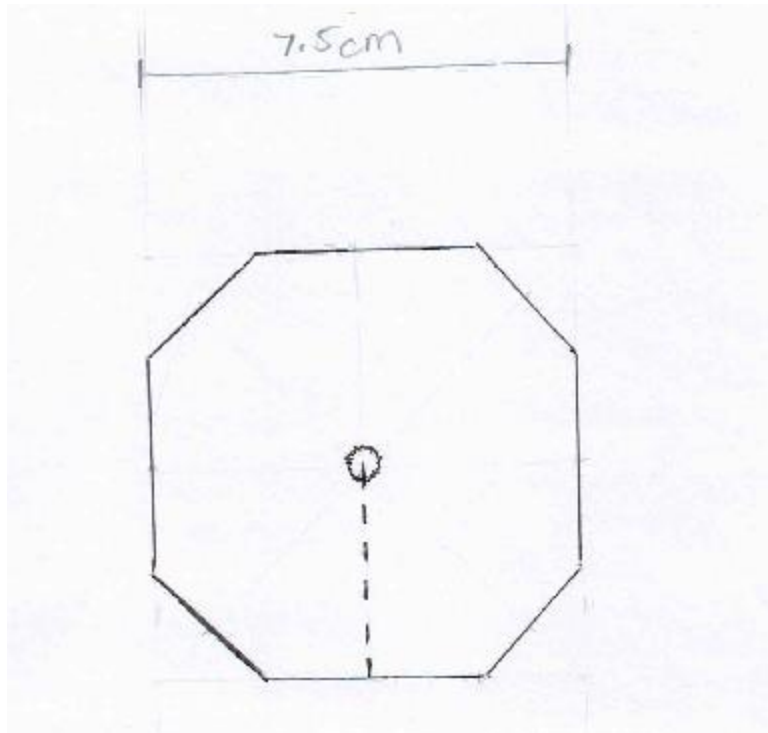

Examples of finished items:

